


Sierra[®]

Ultimate flexibility for a smoother gait

- Angle-top design increases the length of the carbon fiber spring to store and return more energy, and provide a smoother gait
- Split toe design delivers excellent inversion/eversion
- Full-length heel enables foot flat early in the gait cycle, providing a stable base of support and an exceptionally smooth gait
- Rated up to 166 kg (365 lbs)
- 36-month warranty

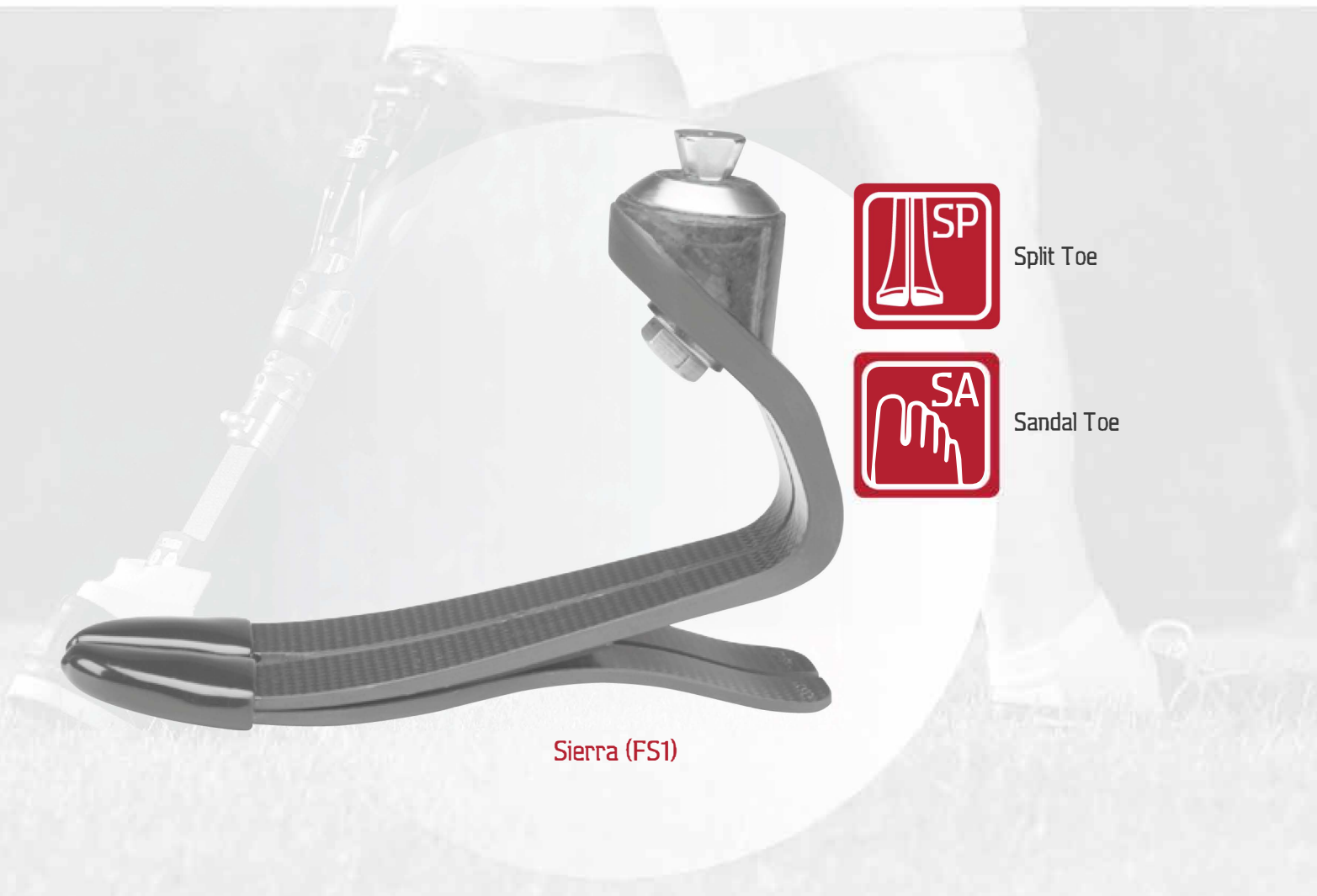

Sierra (FS1)



EVERYDAY

Product ID: FS1

Sizes: 22-31 cm


Sandal-toe available for sizes 22-28 cm

User Weight Rating: 166kg (365 lbs)

Heel Height: 10mm (3/8")

Connector: Male pyramid

Warranty: Foot module 36 months,
Cosmetic foot shell 6 months

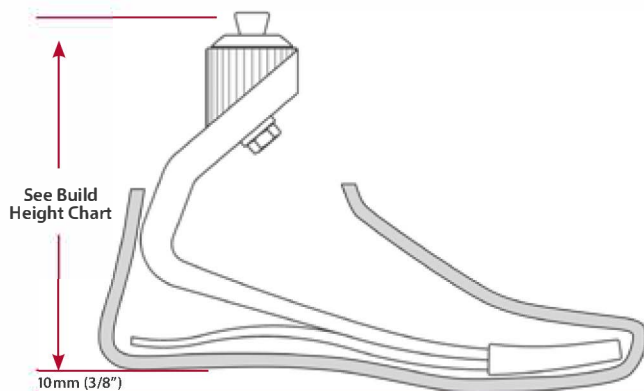

Note: Build height measurements represent a fully assembled unit.

Build Height & Weight

| Size (cm) | Height (in / mm) | Weight (g) |
|-----------|------------------|------------|
| 22-25 | 5¾ / 146 | 18 / 515 |
| 26-28 | 6¾ / 172 | 23 / 640 |
| 29-31 | 7¾ / 187 | 26 / 750 |

Product weight includes foot shell. Measurements are approximate and will vary due to category selection and manufacturing variability.

Stiffness Category Selection

| Weight (lbs) | Impact Level | | | Weight (kg) |
|--------------|--------------|----------|------|-------------|
| | Low | Moderate | High | |
| 100-115 | 1 | 1 | 2 | 44-52 |
| 116-130 | 1 | 2 | 3 | 53-59 |
| 131-150 | 2 | 3 | 4 | 60-68 |
| 151-170 | 3 | 4 | 5 | 69-77 |
| 171-195 | 4 | 5 | 6 | 78-88 |
| 196-220 | 5 | 6 | 7 | 89-100 |
| 221-255 | 6 | 7 | 8 | 101-116 |
| 256-285 | 7 | 8 | 9 | 117-130 |
| 286-325 | 8 | 9 | | 131-147 |
| 326-365 | 9 | | | 148-166 |

Accessories

- Pre-Cut ShinFoam: (twopiece clamshell design)
 - 22-25 cm (AFP-00-FS101)
 - 26-28 cm (AFP-00-FS102)
 - 29-31 cm (AFP-00-FS103)
- Foot Shell Removal Tool: (ACC-00-103)
- Spectra Sock: (ACC-00-101xx) 12", 14", 16"
- Stiffening Bumpers:
 - 22-25cm: (KIT-00-11353-00)
 - 26-28cm: (KIT-00-11354-00)
 - 29-31cm: (KIT-00-11355-00)

Foot Module Part Number

FS1-00-0 **A** -

Category: 1-9 Size: 22-31 Toe Configuration: RU-Regular toe, Universal, SR-Sandal toe, Right, SL-Sandal toe, Left

Foot Shell Part Number

FTC- - **4-**

Generation: 2-3 Profile: M-mid, H-high Cap: 0=yes, 1=no Size: 22-31 Skin Tone: L-light, M-medium, D-dark Toe: R-Regular, S-Sandal Side: L-left, R-right

See the Cosmetic Foot Shell page for more detailed ordering instructions.



FEET